


AVF Dull Needle with Scab Remover

BIOTEQ® AVF Dull Needle with Scab Remover is designed to provide safely and simple removal of the buttonhole scab. The scar tissue tunnel tract allows the needle to pass through to the vessel of the fistula following the same path each time. Available in Twin Packs and Single Packs.



Scab remover are packed closely together with AVF needle, rip off scab remover from one piece PVC bracket before removing the scab.

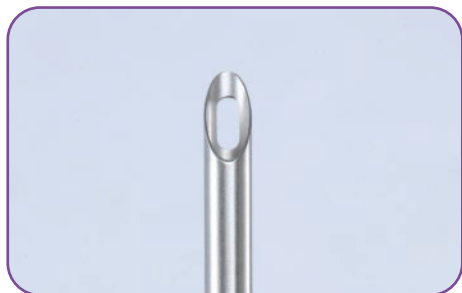

Scab removal using a sharp needle often injures the skin at a buttonhole entry. Buttonhole AVF needles have a blunt needle edge reduces pain and cannulation injury.




STEP 1

Removal of AVF Needle from package.



STEP 2

Rip off scab remover.

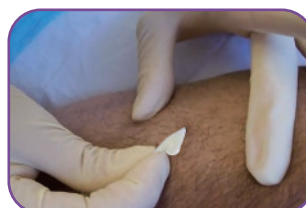

STEP 3

Removing Scab.



STEP 4

Removing Scab.



STEP 5

Cannulating.



STEP 6

Tape securely.